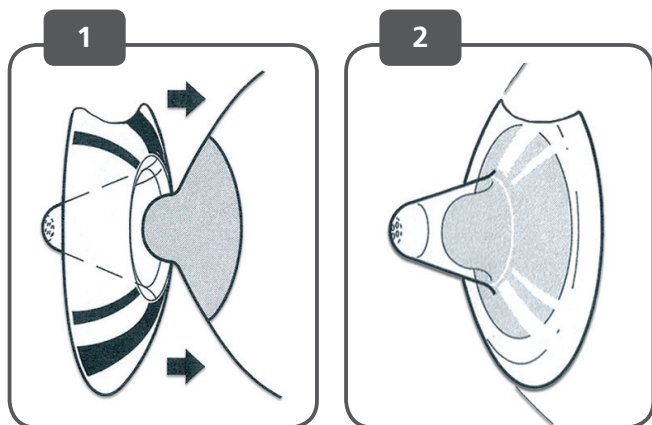
TULIPS STERILE

Instructions for use | Gebrauchsanweisung | Mode d'emploi
Istruzioni per l'uso | Instrucciones de uso | Gebruiksaanwijzing

 ARDO

INSTRUCTIONS FOR USE TULIPS NIPPLE SHIELD

en

-  Consult instructions for use
-  Do not re-use
-  Use-by date
-  Do not use if package is damaged
-  REF Reference number
-  LOT Lot number
-  **STERILE** EO Sterilized using ethylene oxide

Intended Use

Tulips nipple shields help with latching on, inverted or flat nipples and to protect sore nipples during breastfeeding.

Use

- Always wash hands thoroughly with soap and water before handling.
- Moisten the inside of the nipple shield with a little breast milk or water, or use a lanolin nipple cream, so that it adheres more easily to the breast. Pull back the edge of the nipple shield and centre the shield over the nipple (image 1). The baby's nose should be at the tulip-shaped cut-out, so that it is touching the mother's skin.
- Release the edges of the nipple shield onto the breast (image 2).

Notes

- Tulips are available in different sizes. When choosing the correct nipple shield size, the size of the nipple as well as the size of the baby's mouth must be considered.
- Wear only when breastfeeding.
- Sterile packed - intended for single use only.

Disposal

Harmless, in accordance with local regulations.

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