pumpsets and accessories

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Indicações de utilização | Gebruiksaanwijzing | Bruksanvisning | Käyttöohje | Инструкция по эксплуатации | Instrukcja obsługi | Οδηγίες χρήσης | Kullanım Kılavuzu | دليل الاستعمال | 使用说明

Languages**Page**

en	Instructions for use	Ardo Pumpsets and accessories	1–5
de	Gebrauchsanweisung	Ardo Pumpsets und Zubehör	6–10
fr	Mode d'emploi	des Pumpsets Ardo et accessoires	11–16
it	Istruzioni per l'uso	Pumpsets e accessori Ardo	17–21
es	Instrucciones de uso	Pumpsets Ardo y accesorios	22–27
pt	Indicações de utilização	Pumpsets e acessórios Ardo	28–33
nl	Gebruiksaanwijzing	Ardo Pumpsets en accessoires	34–38
sv	Bruksanvisning	Ardo Pumpset och tillbehör	39–43
fi	Käyttöohje	Ardo Pumpsets ja lisävarusteet	44–48
ru	Инструкция по эксплуатации	Наборы для сцеживания и принадлежности Ardo	49–53
pl	Instrukcja obsługi	Ardo Pumpsets i akcesoriów	54–58
el	Οδηγίες χρήσης	Ardo Pumpsets και αξεσουάρ	59–63
tr	Kullanım Kılavuzu	Ardo Pumpsets ve Aksesuarlar	64–68
ar	دليل الاستعمال	والملاحقات التكميلية Ardo طقم المضخة	69–73
zh	使用说明	Ardo 水泵机组及附件	74–78

1. Packaging contents


Pumpset:

- a Adapter tube cover
- b Flexible membrane pot
- c Breast shell 26 mm
- d 1 lip valve plus 1 spare valve
- e Bottle 150 ml (illustration not binding)
- f Tube
- g Tube connector
- h Cap
- i Cap liner

Versions

- A Single Pumpset
- Double Pumpset
- Clinic Pumpset
- Pumpset Ardo Alyssa & Bellis
- B Pumpset Ardo Alyssa


WARNING: Danger of suffocation – contains small parts. Keep small parts away from babies and small children.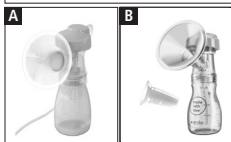

2. Supplementary accessories

- j 36 mm breast shell
- k 31 mm breast shell
- l 28 mm breast shell insert
- m 22 mm breast shell insert
- n 28 mm breast shell insert, silicone
- o 22 mm breast shell insert, silicone
- p 18 mm breast shell insert, silicone
- q Optiflow breast shell insert
- r Bottle holder
- s Bottle stand

The choice of two Ardo breast shells and breast shell inserts enables you to find the size that is right for your nipples. The Ardo Optiflow massage insert guarantees even gentler expressing and the massage effect encourages the flow of milk (can only be used with the 31 mm breast shell). The bottle holder is compatible with the Calypso Double Plus breastpump and the bottle stand is compatible with Calypso and Ardo Alyssa.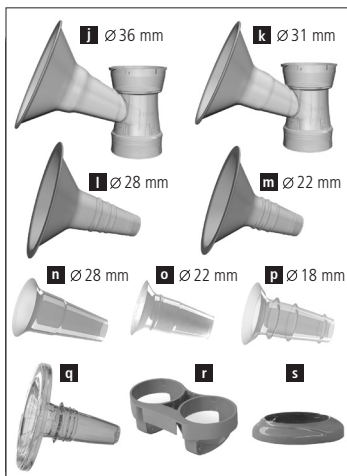

3. Intended use

The Ardo pumpset should be used in combination with the Ardo breastpumps and is intended for the expression, collection and storage of breast milk. The pumpset can be used as a single or double pumpset.

4. User risk

There are no known contraindications.


Safety regulations: Only use the pumpset if it is clean (Sections 6 and 7). Check the pumpset for damage after cleaning.

According to Regulation (EU) 2017/745 on medical devices, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

5. Assembling the pumpset

Assemble as illustrated. Attach the adapter tube cover so that both marks match up and then rotate to the right.

Important: Boil before first use (see the section 'Cleaning at home') and check whether the flexible membrane pot is intact with the smooth side facing outwards.

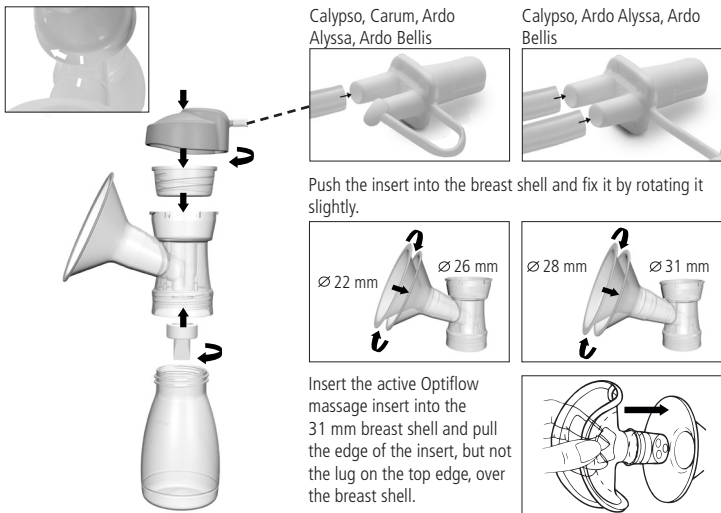6. Cleaning at home

Manual cleaning

Before using for the first time and after each time it is used:

- Dismantle the pumpset into its individual parts.
- Rinse for at least 1 minute using cold tap water.
- Clean the components of the pumpset with commercial detergent and warm water (not over +40°C / 104 °F) and rinse with cold tap water. Make sure that all detergent residues are removed.
- A soft brush can also be used for heavy soiling.

ATTENTION! Do not clean the lip valve with the brush.

- Allow all parts of a clean surface to air dry.

Manual disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process. If you discover any changes, dispose of the part affected and replace with a new one.

The illustrated parts can be wiped down with a cloth as needed. It is not necessary to flush the tube.

Do not boil the bottle holder.

7. Cleaning in hospitals (only for the Clinic Pumpset)


Sterilise the disassembled pumpset at 121°C (250°F) for 20 min / 134°C (273°F) for 5 min.

8. Use

Wash your hands thoroughly. Sit in a comfortable and relaxed position.


8.1 Expressing with one pumpset

- Connect the pumpset to the Ardo breastpump.
- Centre your nipple so that it moves freely in the breast shell (breast shell insert) when pumping.
- Switch the breastpump on.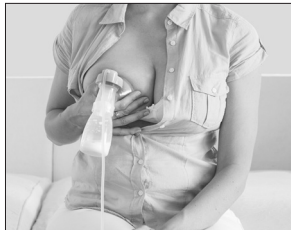


8.2 Expressing with a double pumpset

- Connect the second pumpset to the Ardo breastpump.
- Procedure for the **Ardo Carum**, **Ardo Alyssa** and **Ardo Bellis** breastpumps: Place a pumpset on the breast, switch the pump on and place the second pumpset on the other breast. Alternatively, you can also attach both pumpsets at the same time.
- Procedure for the **Ardo Calypso** breastpump: Attach both pumpsets at the same time (see illustration). Switch the pump on and hold each pumpset separately in your hands.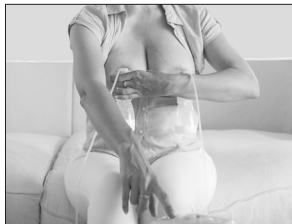

9. Choosing a breast shell and Optiflow massage insert

If your nipple rubs uncomfortably on the inside of the funnel, use a larger breast shell (breast shell insert). If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow can only be used with the 31 mm breast shell. Store the Optiflow insert protected from light (silicone).

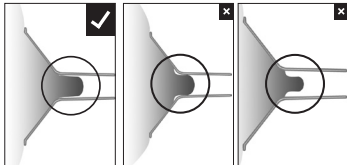10. Storing breast milk

Recommended storage time at room temperature: up to 4 hours at 25°C (77°F); in the refrigerator: up to 4 days at 4°C (40°F); in the freezer compartment with door: 6 months up to 1 year at -18°C (0°F).

ATTENTION: Do not put more than 150 ml of breast milk in the bottle because milk expands when it freezes. Do not heat breast milk in the microwave.

11. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the pumpset is correctly assembled and is not damaged.
- Check if the lip valve is cracked or stuck together.
- Check if the flexible membrane pot is intact with the smooth side facing outwards.
- Ensure that the adapter tube cover forms a tight seal.

Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size is right for your nipples.
- Check that the vacuum intensity is not set too high.
- Contact a professional for help.

If you have problems with breastfeeding, please contact your midwife, lactation consultant or doctor.

12. Warranty

There are legally prescribed minimum warranty periods from the date of purchase that are specific for each country. The warranty coverage is limited to the first purchaser.

13. Spare parts

63.00.267 Lip valves (2 pieces)

63.00.269 Flexible membrane pot, adapter tube cover (2 pieces)

14. Technical information

Service life of the pumpset: As long as the pumpset is not faulty, it can be used. Please check the pumpset for damage before every use.

	Compliance with the valid European regulations for medical devices		Manufacturer
	Follow instructions for use		Follow instructions for use
	Medical device		Pumpset is a BF type applied part
	Materials: Polypropylene (PP) / silicone / ABS		Relative humidity: 20%–95%
	Temperature: Operation +5°C (41°F) to +40°C (104°F) / Transport -20°C (-4°F) to +50°C (122°F)		Air pressure: 700–1060 hPa
	Dispose of with polypropylene		Protect packaging from sunlight
	Packaging: Caution, fragile		Store in a dry place

15. Disposal

Waste must be disposed of in accordance with local regulations.

The German text is the reference for the contents.

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Ardo medical AG