EN ASPIRATION SYSTEM CANISTER

Please read this manual carefully before using the product, and do not perform any actions other than those specified in this manual. It can only be used as instructed and as specified by specialist clinicians. Strictly do not use it except instructions. It is the responsibility of each medical facility to ensure that trained, competent and knowledgeable personnel about the processes and the hospital infection control protocol are involved in the use of these medical devices. Pergo cannot be held responsible if the product is used differently than described in these instructions. Using products whose integrity is not ensured is dangerous and forbidden. Use the device before the 'Use By' date as specified on the product label.

INTENDED USE

Aspiration System Canister is designed to hold the Suction Liner Bag, and provide the necessary connections to the compressor, patient, and other canisters when serial connection of multiple canisters are made.

PACKAGE CONTENTS

- 1- Canister(reusable)
- 2- Vacuum Connector(attached to the Canister)

USAGE OF THE PRODUCT

- a- Place the canister in a bracket (rail or wall) in an upright position as shown in Figure 1.1.
- b- The distance between the vacuum source and the vacuum connector should be measured and the silicone hose must be cut 1.5 times the measured distance as shown in Figure 1.2.
- c- One end of the cut hose should be attached to a vacuum source as shown in Figure 1.3 and the other end should be attached to vacuum connector on the canister.

Note: There is a scale that shows the amount of liquid aspirated on the canister and scale is sensitive to ± 100 mL.

Note: The canister can also be attached to the Pergo Aspiration system transport cart.

WARNINGS

- The product is designed for the use of specialist doctors or healthcare professionals.
- Inspect the product for damage detection before using it.
- Any damage to the product (eg, bent, bent, deformed, or cut) may affect product performance.
- Precautions should be taken for the risks that may occur due to the patient.
- Keep a spare product in case of error.
- The user should take precautions against the risk of contamination and use it carefully.
- Do not go beyond the intended use.

NOTE: Any negative situation regarding the product should be reported to the manufacturer or authorized dealership.

INDICATIONS

There are no indications.

CONTRAINDICATIONS

There are no contraindications.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations. The product is re-usable.

STORAGE AND TRANSPORTATION CONDITIONS

Products should be stored in a closed warm and dry environment. All products should be protected from damp, dirty and dusty environments. Products can be transported under ambient atmospheric conditions provided that they are protected from rain, dust and direct sunlight. Temperature-controlled transportation is not required.

SHELF LIFE

This product has no shelf life limitation.

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